

Rückenlage



ProAxis® – Supine Patient Positioning



Users must read and understand the ProAxis® Surgery Table Owner's Manual before using this device. Instructions provided are for suggested setups. Final determination of patient positioning and setup is at the discretion of the surgeon and surgical team. Each number in the setup image identifies the component(s) described in the corresponding step in the following setup instructions.

Safety straps must be used on patient at all times.

ProAxis® – Supine Patient Positioning

1. Table Preparation and Connecting IntelliPendant®



Roll the table to where it will be used and orient the Head-End of the table in the room where the anesthesiologist work station is located. Plug the power cord of the ProAxis® Table into a properly grounded receptacle. Connect the IntelliPendant® to the port on the Head-End column. Press the Power Switch and observe that the switch illuminates with a green ring indicating that power is applied to the table.

2. Head-End Supine Top



To install the Head-End Supine Top on the frame, open the Locking Brackets, align the Locking Brackets over the rails, and lower into place. Slide the Top toward the Head-End Column until it rests against the Brackets of the rail frame on both sides and lock into place.

3. Cable Connector



Connect the Cable Connector to the port on the Head-End Assembly and turn the locking collar.


4. Foot-End Supine Top



To install the Foot-End Supine Top on the rail frame, open the Locking Brackets, align the Locking Brackets over the rails, and lower into place.

Slide the top toward the Foot-End Column until it rests against the Brackets of the rail frame on both sides and lock into place.

5. Tempur-Pedic® Medical Pad

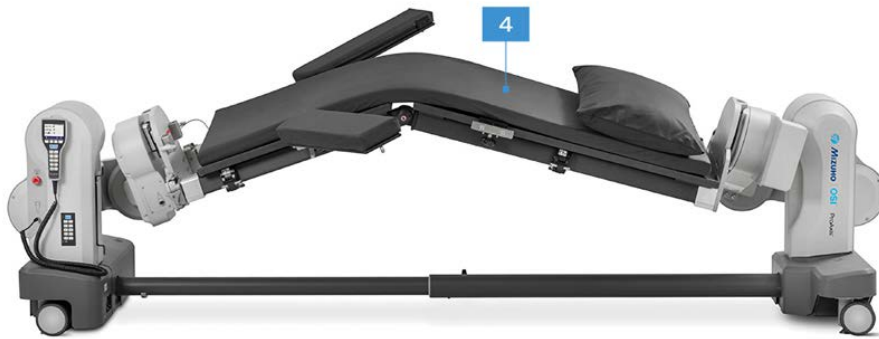

Install the Tempur-Pedic® Medical Pad. Align the “A” label on the bottom of the pad over the corresponding label on the Head-End Supine Top. The Velcro® on the pad and the top should also be aligned. Lower the pad onto the top securing it with the Velcro®.

6. Patient Instructions



Transfer the patient in supine position and secure with patient safety straps.

7. Mounting Components or Accessories



Use the Universal Side Rail Adapter when mounting accessories and components such as arm boards.

8. Optional - Universal Side Rail Adapter



To install the Universal Side Rail Adapter, open the clamp using the black knob and place the Adapter in the desired location with the black knob facing downward. Tighten the knob to secure it in place.



9. Optional - Pivoting Arm Board



Hold the locking lever of the Pivoting Arm Board in the open position and place over the edge of the Universal Side Rail Adapter. Lower the Arm Board into place and release the lever.

To pivot, release the latch by pulling it, and pivot the Arm Board to the desired position.



10. Optional - Tempur-Pedic® Medical Pillows

Pillows may be used under the patient's head and legs to assist with final positioning, stability and comfort.



11. The Supine Patient Positioning setup is now complete.

For more information or questions about this setup or any setup, please refer to the Owner's Manual or contact Mizuho OSI for additional support.

