

Seitenlage



ProAxis® – Lateral Patient Positioning



Users must read and understand the ProAxis® Surgery Table Owner's Manual before using this device. Instructions provided are for suggested setups. Final determination of patient positioning and setup is at the discretion of the surgeon and surgical team. Each number in the setup image identifies the component(s) described in the corresponding step in the following setup instructions.

Safety straps must be used on patient at all times.

ProAxis® – Lateral Patient Positioning

1. Table Preparation and Connecting IntelliPendant®



Roll the table to where it will be used and orient the Head-End of the table in the room where the anesthesiologist work station is located. Plug the power cord of the ProAxis® Table into a properly grounded receptacle. Connect the IntelliPendant® to the port on the Head-End column. Press the Power Switch and observe that the switch illuminates with a green ring indicating that power is applied to the table.

2. Head-End Supine Top



To install the Head-End Supine Top on the frame, open the Locking Brackets, align the Locking Brackets over the rails, and lower into place. Slide the Top toward the Head-End Column until it rests against the Brackets of the rail frame on both sides and lock into place.

3. Cable Connector



Connect the Cable Connector to the port on the Head-End Assembly and turn the locking collar.


4. Foot-End Supine Top



To install the Foot-End Supine Top on the rail frame, open the Locking Brackets, align the Locking Brackets over the rails, and lower into place.

Slide the top toward the Foot-End Column until it rests against the Brackets of the rail frame on both sides and lock into place.

5. Tempur-Pedic® Medical Pad

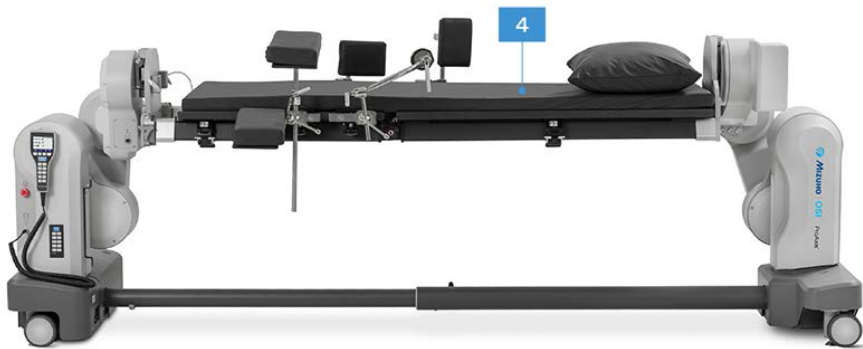


Install the Tempur-Pedic® Medical Pad. Align the “A” label on the bottom of the pad over the corresponding label on the Head-End Supine Top. The Velcro® on the pad and the top should also be aligned. Lower the pad onto the top securing it with the Velcro.

6. Mounting Components or Accessories



Use the Universal Side Rail Adapter when mounting accessories and components such as arm boards.


7. Universal Side Rail Adapter

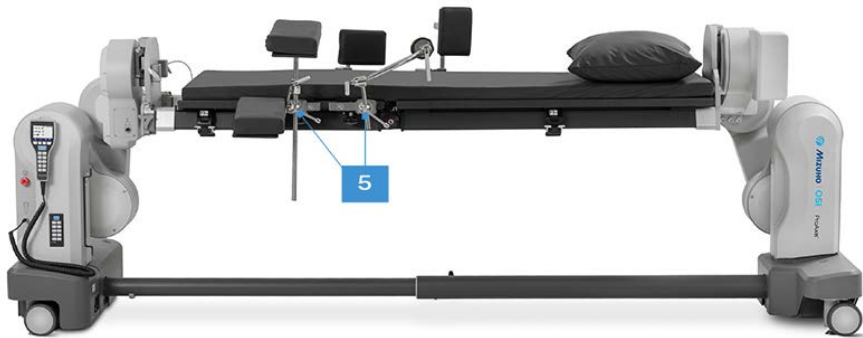

To install the Universal Side Rail Adapter, open the clamp using the black knob and place the Adapter in the desired location with the black knob facing downward. Tighten the knob to secure it in place.



8. Clark Socket

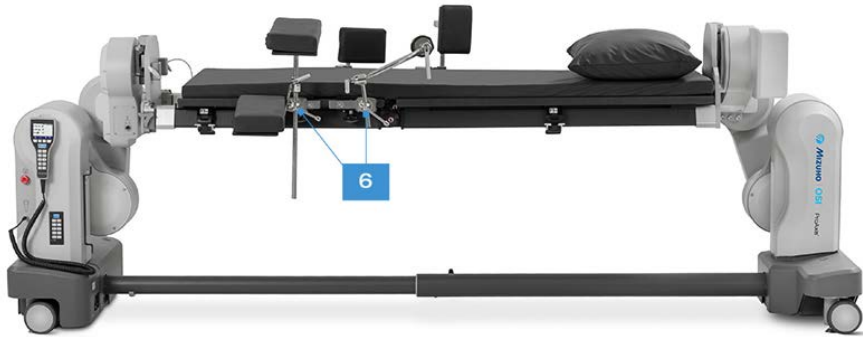

Slide the Clark Socket onto the Universal Side Rail Adapter.
Do not tighten until accessory post is inserted.



9. Patient Instructions



Transfer the patient in supine position. Stabilize and maintain the patient in the lateral decubitus position using safety straps or a lateral positioning device.

10. Lateral Arm Board Unit



Insert the post of the Lateral Arm Board Unit into the Clark Socket, and tighten the Clark Socket to secure the post.




11. Installation Tip



To adjust the position of the lower Arm Board, release the Drop Handle and pivot the Arm Board into place. Tighten the Drop Handle.


12. Aluminium Side Rail Sockets

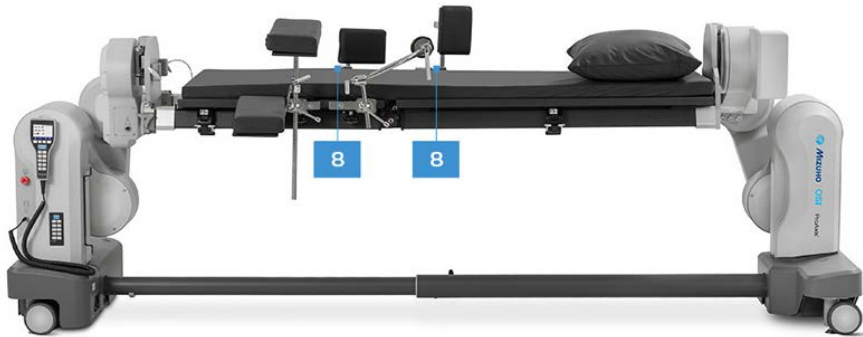


To mount the rectangular Lateral Positioners, place an Aluminium Side Rail Socket on the Universal Side Rail Adapter. Do not tighten the T-Handles on the Aluminium Side Rail Socket.




13. Lateral Positioners

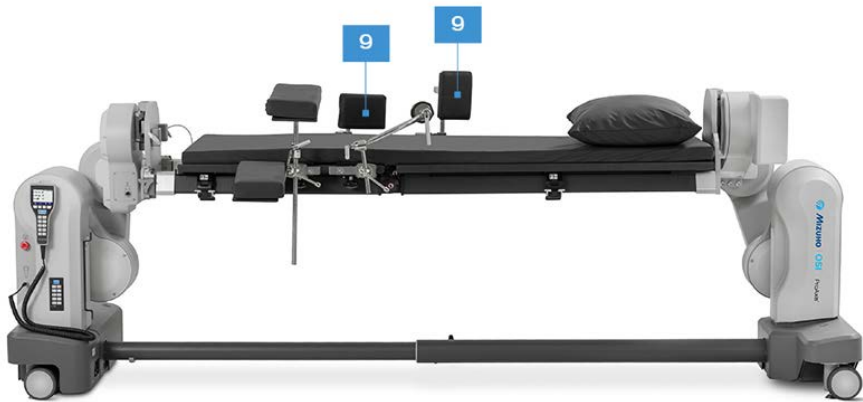

Mount the rectangular Positioners such that one pad is between the scapulae, the second at the sacrum, and the third over the pubis. Tighten the T-Handles of the Aluminum Side Rail Sockets. Adjust each pad position until it rests against the patient.

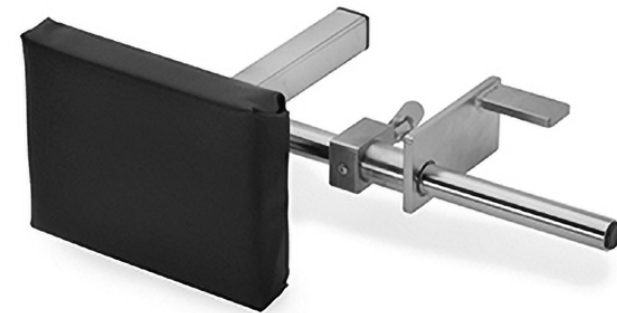

14. Deluxe Lateral Positioners

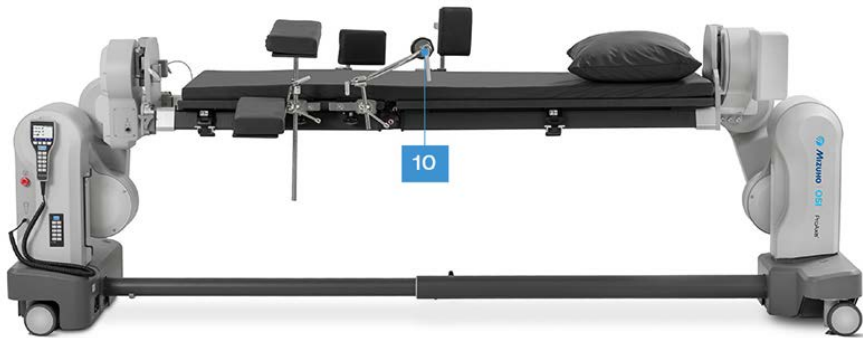

The Deluxe Lateral Positioner Set features two rectangular pads and a Pubic Pad.

To install the Pubic Pad, mount a Clark Socket on a Universal Side Rail Adapter, and place the post of the Pubic Pad in the Clark Socket. Tighten Clark Socket to secure.



15. Installation Tip



To adjust the Pubic Pad, loosen the Drop Handle, pivot to the desired position, and lock in place.

16. Optional - Pivoting Arm Board



Hold the locking lever of the Pivoting Arm Board in the open position and place over the edge of the Universal Side Rail Adapter. Lower the Arm Board into place and release the lever.

To pivot, release the latch by pulling it, and pivot the Arm Board to the desired position.



17. Optional - Tempur-Pedic® Medical Pillows



Pillows may be used under the patient's head and legs to assist with final positioning, stability and comfort.



18. The Lateral Patient Positioning setup is now complete.

For more information or questions about this setup or any setup, please refer to the Owner's Manual or contact Mizuho OSI for additional support.

